

# NON N-95 MASKS

For clarity, these are the four Non N-95 masks that AUMC regularly stocks. We may have additional masks secondary to outside donations related to COVID-19. Non N-95 masks are required in high risk areas.

Optional in low risk areas.

Visitors will be required to wear a surgical or cone mask while in the hospital.




**Reuse Instructions:** Store in the brown paper bag during your shift per the instructions.

| Mask Type                                                                                                                                               | Mask Type                                                                                                                                                                                                                                               |
|---------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p data-bbox="342 674 597 758">Isolation Mask<br/>(yellow)</p>        | <p data-bbox="980 674 1377 806">Halyard Brand Duckbill<br/>SURGICAL Mask<br/>(with ties)</p> <p data-bbox="954 814 1403 846">Warning: This is not an N-95 mask.</p>  |
| <p data-bbox="204 1398 732 1440">Surgical Mask with Face Shield</p>  | <p data-bbox="1081 1388 1276 1423">Cone Mask</p>                                                                                                                    |

# N-95 RESPIRATOR MASKS

For clarity, N-95 respirator masks are designated for specific patients on Airborne Isolation or Emerging Pathogens Precautions.

**Required for aerosol-generating procedures and treatments for COVID-19 Positive patients or Patients Under Investigation (PUI).**

| Mask Type                                                                                                                                                                                                                                                                                                                           | Re-use Instructions                                                                                                                                                                                        |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>Halyard Brand N-95 Respirator<br/>(duckbill with elastic straps)</b></p> <p>Warning: Please note the Halyard N-95 duckbill respirator has elastic straps and should not be confused with the Halyard non N-95 duckbill mask with ties.</p>  | <p>Store in the brown paper bag during your shift per the instructions.</p> <p>Halyard brand N-95 respirators may not be reprocessed.</p> <p><b><u>DISCARD THIS MASK AT THE END OF YOUR SHIFT.</u></b></p> |
| <p><b>3M N-95 Respirator</b></p> <p>Small                      Regular</p>                                                                                                                                                                       | <p><b><u>3M N-95 Respirators are to be sent for reprocessing if the mask is in good working condition. Follow instructions for your unit.</u></b></p>                                                      |
| <p><b>N-95 Respirator (HDX Brand-Home Depot)</b></p>                                                                                                                                                                                             |                                                                                                                                                                                                            |